

# IMPLANT SET PREMIUM



Pat. Pending



# IMPLANT SET PREMIUM

Our new packaging brings you all you need for the implant procedure.  
You must choose the implant, CLASSIC or DYNAMIC.



You can choose the diameter easily by colored code.

The Length is written at the top of the capsule cap.



Open the external tube and take out the internal tube.




Take out the implant by the mount/transfer from the internal tube and handily drive it into the surgical bone preparation.  
You can drive it with a ratchet too.



Separate the mount and the transfer if you need the transfer now or save it for later use.



On the other side of the tube, you can choose closing the implant by using the "cover screw" for two stage surgery.

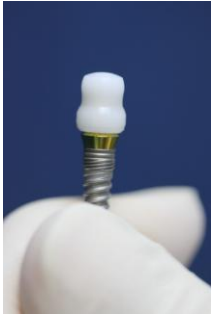

Or you can choose using the abutment and the plastic healing cap for one stage surgery .

# IMPLANT SET PRIME



In this innovate packaging you'll get only the implant and the mount for handily into the surgical preparation.



Open the external tube and take out the internal tube.



Take out the implant by the mount from the internal tube and handily drive it into the surgical bone preparation.  
You can drive it with a ratchet too.

The future is here...

A handwritten signature in blue ink, appearing to read 'Shmuel Rozen', is written over a light blue background.

Shmuel Rozen

VP marketing

Cortex Dental Implants Ltd.